

Using the Standard E-Sac

The large capacity of the sac and the security tail are clearly visible.

Note the position of the loops on the bottom and on the mouth of the sac. Use them to hold the sac during deployment and capture of tissue to be removed.

Pass a grasper through the loop on the mouth and take hold of the bottom loop.

If your access cannula has no reducing valve, first withdraw the tail through a 10/5mm reducer, then pass the grasper through the valve, and finally pull the reducer up the shaft of the grasper

Push the sac into the access cannula, and stop before the tab on the mouth of the sac disappears. Release the grasper and take hold of the mouth tab.

. You are now ready to deploy the sac into the abdomen.

If you are using a reducer, slide it down the shaft of your grasper to engage the access cannula and prevent the loss of gas.

Deployment is completed by pushing the grasper through the access cannula until the mouth is fully opened in the abdomen, while at the same time holding the tail of the sac in tension against the shaft of the grasper.

